

Aries Work Week Barrier Face Covering (BFC)

Ultra Filtration...Maximum Breathability




Purpose	Used as source control for members of the general public, as well as health care personnel (HCP) in healthcare settings, to cover their noses and mouths, in accordance with CDC recommendations, to help prevent the spread of the SARS-CoV-2 during the COVID-19 pandemic.
Form Factor	Flat-fold, pleated design Designed to last for a 5-day, 40-hour work week Packaged 5 per pack: 1 package / per user / per month
Materials	85% polypropylene 15% Ingeo™ Biopolymer (sustainable polymer with lower carbon footprint) Patented polyester/spandex ear loops
Size	One Size fits most
Reuse	Each BFC is designed to last for a typical 40-hour work week
Manufactured	All materials are made and assembled in the USA




Filtration and Breathability:

The new ASTM standard helps companies make informed decisions on the masks they are providing employees, including what filtration levels they need to provide for maximum safety and how much breathability they are delivering. To solve the challenges of both filtration and breathability, Aries worked with leading providers of filtration polymers and created Captur™, a novel, engineered fabric technology that provides high levels of protective filtration combined with maximum breathability. Aries barrier face coverings deliver Level 2, High Performance for both standards.

Sub-micron particulate filtration efficiency
Effectiveness of barrier face covering for capturing small particles from wearer; larger percentages indicate higher performance

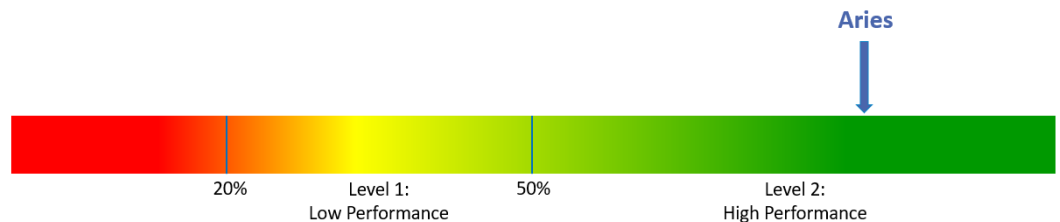

Air-flow resistance (Breathability)
Indicative of ease of breathing while wearing barrier face covering; lower resistances indicate more breathable products

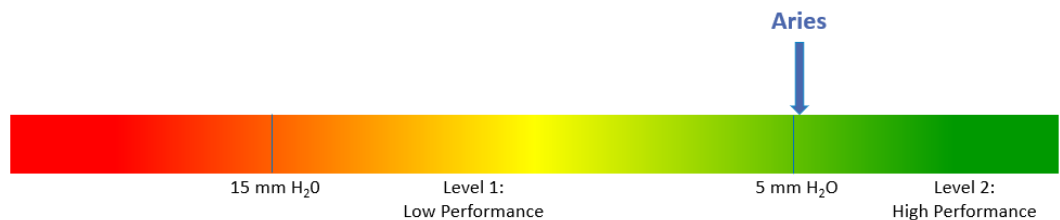

The Aries Work Week Barrier Face Covering (BFC) is intended for to be used for one work week. Manufactured with Captur™, a novel, engineered fabric technology that provides high levels of protective filtration combined with maximum breathability.

INSTRUCTIONS FOR USE:

PUTTING ON YOUR BFC

WASH HANDS OR USE AN ALCOHOL-BASED HAND SANITIZER IMMEDIATELY BEFORE PUTTING ON PPE

- BFC can be worn on either side. Best worn with labeling facing outward.
- Secure elastic bands behind ears.
- Use 2 fingers to pinch the noseband to fit tightly to nose.
- Fit snug to face, ensuring coverage of both nose and chin

REMOVING YOUR BFC

WASH HANDS OR USE AN ALCOHOL-BASED HAND SANITIZER IMMEDIATELY BEFORE REMOVING PPE

- Remove BFC touching only the elastic ear loops. Grasp the ear loops and remove without touching the front of the BFC.
- After 40 hours or 5 uses, discard in a waste container.
- Rewash hands after handling BFC.

CAUTIONS

- DO NOT touch or adjust your BFC without cleaning your hands before and after use.
- DO NOT wear your BFC solely under nose or mouth.
- DO NOT wear your BFC around your neck or on top of head.

STORAGE BETWEEN USES

- Store at room temperature in a clean, dry place, away from sunlight.
- Avoid contact with other surfaces or other PPE.
- Unused BFCs should be left in their original packaging or placed in a resealable plastic bag.
- After 5 days or 40 hours of wear, discard in a waste container.

DISINFECTING BETWEEN USES (OPTIONAL)

- BFCs are designed to be worn for 5-day, 40-hour work week.
- If desired, and as needed, you may disinfect:
 - Use 3% Hydrogen Peroxide solution in a spray bottle
 - Spray both sides of mask and allow the solution to evaporate (approximately 6-8 minutes)
 - Hang to fully dry
- If the BFC is visibly soiled, discard and replace with another BFC.

REGULATORY COMPLIANCE

- Meets ASTM F3502 specification for Barrier Face Coverings. This product is primarily intended as a means of source control for minimizing the projection of expelled materials from the wearer's nose and mouth.
- WARNING: This Barrier Face Covering is not a medical mask as defined in ASTM F2100, is not intended for use in medical procedures, and is not a respirator.
- Barrier face coverings should not be placed on very young children (<2 years) or anyone who has trouble breathing, or anyone who is unconscious, incapacitated, or otherwise unable to remove the barrier face covering without assistance.



SALES

sales@aries-usa.com

ABOUT ARIES

At Aries, we believe that wearing a mask should not be an unpleasant experience. What good is high filtration capacity if you are constantly removing or adjusting your mask throughout the day to breathe? Aries Masks are manufactured from Captur, a novel, engineered fabric technology providing high levels of protective filtration combined with breathability you must try to believe. We do not just want to meet published filtration and breathability standards; we want to be the standard others are judged by.

©Aries USA, Inc. All rights reserved. Aries, the Aries logo and other marks appearing herein are property of Aries USA, Inc. and/or one or more of its subsidiaries and partners. All other marks are the property of their respective owner(s).